



DPMG Client Update:

DPMG is now offering rectal and throat Chlamydia and Gonorrhea testing using the Aptima Multitest Swab!!!

COLLECTION

1. Use the APTIMA® Multitest Swab. (please see collection guide).
2. Use a separate swab if ordering other STI's from additional sources. Additional tests should not be ordered on the same swab.
3. Please label the vial "Throat" or "Rectal", especially if submitting an additional vial for additional STI testing.
4. The specimen is viable for 30 days from collection.

ORDERING

If using the DPMG cytology requisition, please choose the desired test and write in "throat" or "rectal". If using an EPIC generated requisition, please write the source next to the CT and/or NG order. See examples:

DPMG Requisition

MOLECULAR TESTING	
Pap Vial, Aptima Swab or Urine Transport Tube	
<input checked="" type="checkbox"/> Chlamydia	Throat
<input type="checkbox"/> Gonorrhea	
<input type="checkbox"/> Trichomonas	

EPIC Requisition

Comments

Chlamydia testing (Pap vial, Aptima swab, or Aptima urine)

Source: Throat

For more information or for supplies, please contact your Client Liaison or Customer Service at 916-446-0424.